



# TBP: App Cheat Sheet

## TAKING TOTAL BODY IMAGES


1 Tap the icon

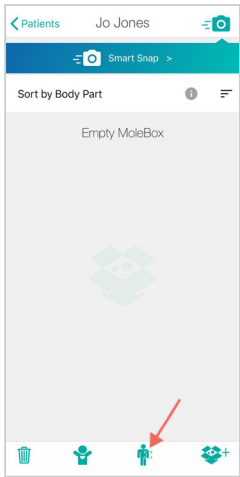

2 Tap Serial Total Body Photography or select the body part you want to image

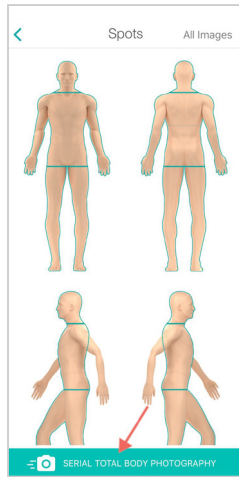


3 Use baseline ghost as a guide

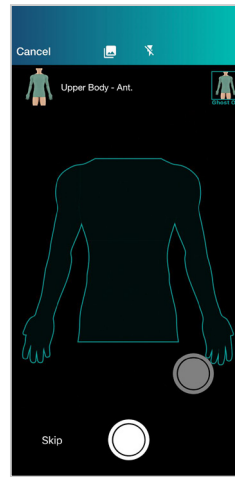


## LESION DETECTION

1 Select Body Part

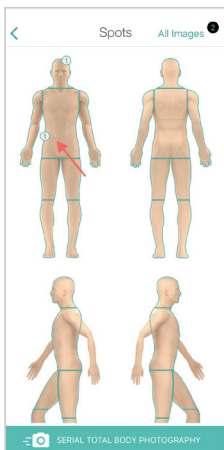


2 Select Image

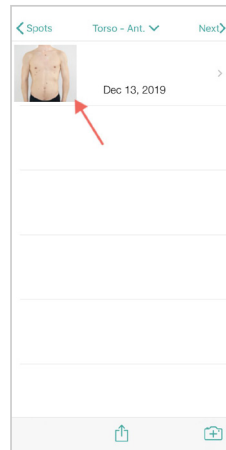

3 Tap on the icon




4 Select Lesions of Concern and press Next

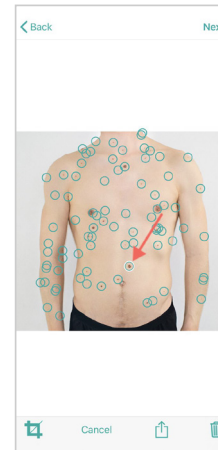


# DERMOSCOPIC IMAGING AND BODY IMAGING

1 Attach your Dermoscope and take image

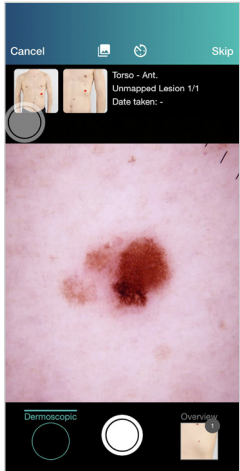

2 Map on Body Map and press Place

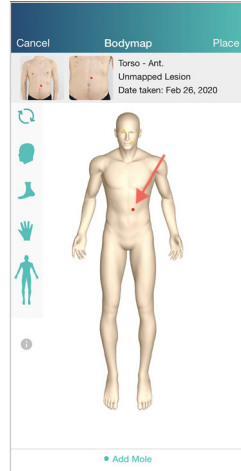

3 Take more images or press Next/Done

