



TBP: Web Cheat Sheet

TAKING TOTAL BODY IMAGES



1 Tap the icon

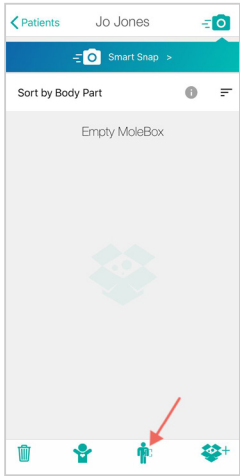

2 Tap Serial Total Body Photography or select the body part you want to image

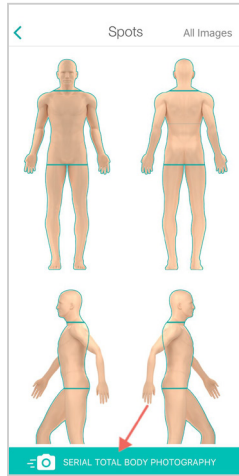


3 Use baseline ghost as a guide

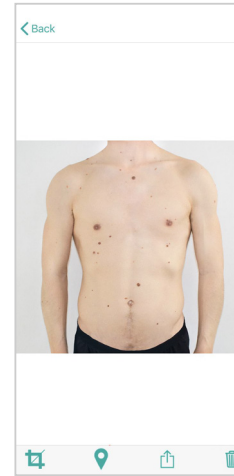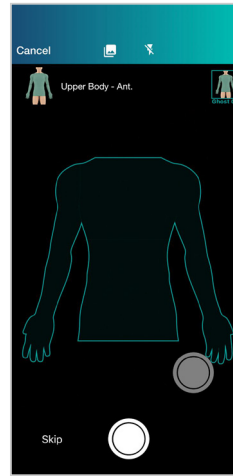


MAPPING ON THE WEB


1 Select Patient -> Total-Body

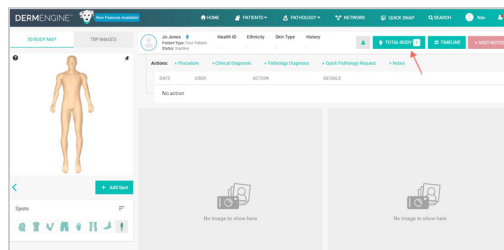


2 Select Lesion(s) of Concern and click Start Mapping

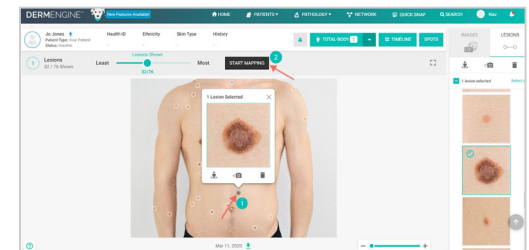

3 Place corresponding spots on body map and Save to body map

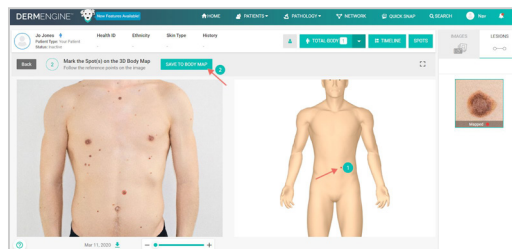


4 Select Take Images on App and select device

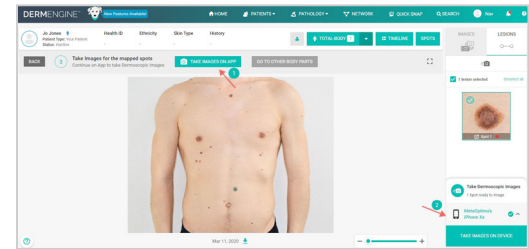


DERMOSCOPIC IMAGING

1 Open Notification on App

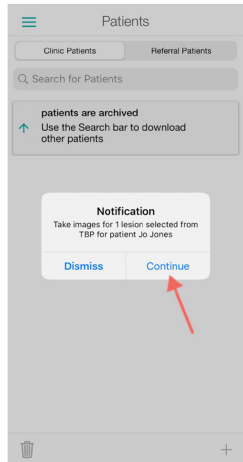

2 Attach your Dermoscope and take images

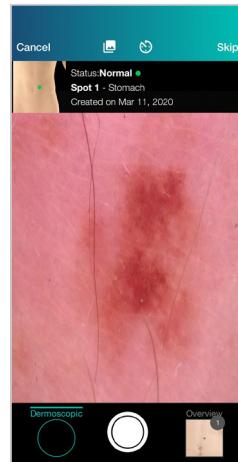

3 Take more images or press Next/Done

