

METAOPTIMA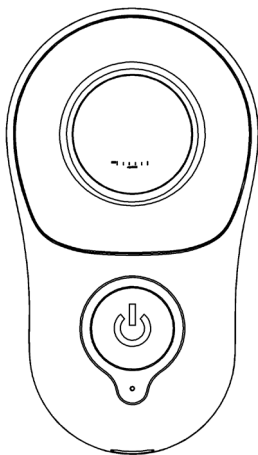

User's Manual

MoleScope II and TrikoScope

P002 - 004 - IFU
2023-06-06 V3.2

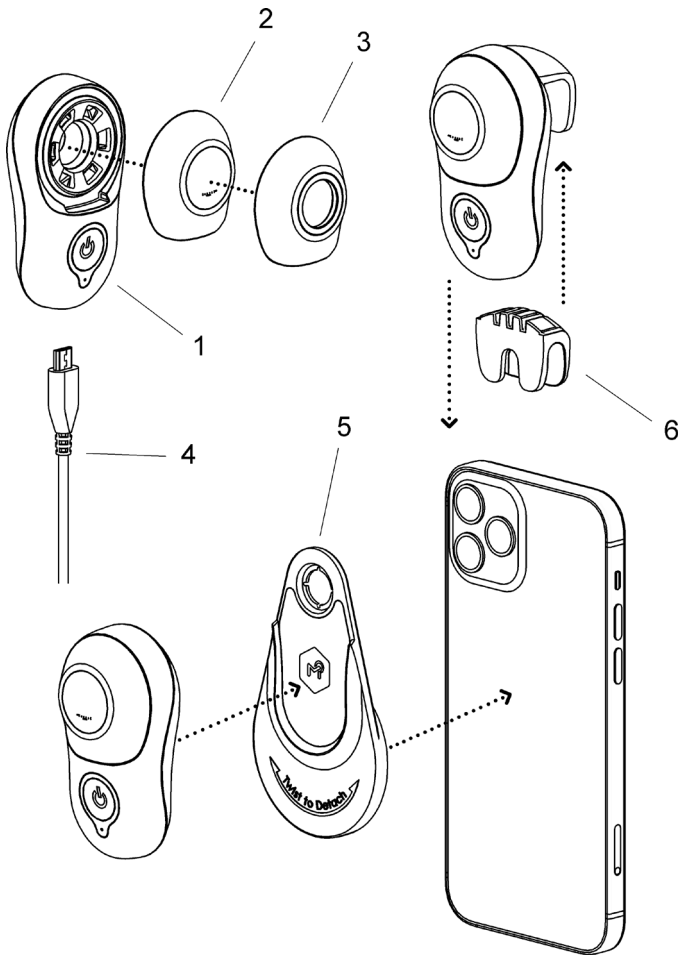#	EN: Component Name DE: Komponente FR: Nom du composant ES: Nombre del componente IT: Nome del componente NL: Naam onderdeel
1	Mobile Dermatoscope Mobil-Dermatoskop Dermatoscope mobile Dermatoscopio móvil Dermatoscopio mobile Mobiele dermatoscoop
2	Contact Cap Kontaktkappe Capuchon de contact Tapa de contacto Cappuccio di contatto Contactkapje
3	Non-Contact Cap erührungslose Kappe Capuchon sans contact Tapa sin contacto Cappuccio non di contatto Niet-contactkapje
4	USB Charging Cable USB-Ladekabel Cabine de chargement USB Cable de carga USB Cavo di ricarica USB USB-oplaadkabel
5	Universal Attachment Plate (for Model U) Universal-Befestigungsplatte (Modell U) Plaque de fixation universelle (pour le modèle U) Placa de acoplamiento universal (para el modelo U) Piastra di attacco universale (per il modello U) Universele bevestigingsplaat (voor model U)
6	Rubber Attachment (for Models A, A+, B, B+) Gummiaufsatz (Modelle A / A+ / B / B+) Fixation en caoutchouc (pour les modèles A, A+, B, B+) Accesorio de goma (para modelos A, A+, B, B+) Attacco di gomma (per i modelli A, A+, B, B+) Rubberen bevestigingsstuk (voor model A, A+, B, B+)

Please read this user manual carefully before using the device.

Intended Use

MoleScope is a battery-powered dermatoscope intended to be attached to smartphones and tablets to take images of skin, scalp, hair, nail, and other body surfaces with high magnification and cross-polarized light. It is a reusable, non-invasive device for imaging and documentation only.

Indications of Use and Contraindications

The device is indicated for use in combination with a smartphone or a tablet when magnified images of the skin, scalp, hair, nail, and other body surfaces of any patient demographic are required. The device should not be used on open wounds or on any other sensitive areas. The device is intended to be used by professional healthcare workers or lay persons.

Cautions

- Do not look directly into the bright LED light. Eyes must be closed during facial examination.
- Do not attempt to open the device for any reason.
- Do not attempt to change the device battery. The battery is not replaceable.
- Do not use the device to image sensitive areas, open wounds, and surfaces near the eyes, ears, nose, and mouth.
- Check your device before use if it has been dropped.
- Follow the instructions in this document to clean and disinfect contact and non-contact caps in between patients.
- Keep the device and detachable components out of reach of children and pets. If swallowed, seek medical attention immediately.
- Do not use the device while it is in charging mode.
- This device contains magnets. Do not use it without first consulting your doctor if you wear a pacemaker or other medical implant.

Consult a healthcare professional to determine your skin assessment plan.

For EU/UK Residents

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established or in case of UK residents, to the manufacturer and to the secretary of State in the United Kingdom.

Components Include


Device Models A / A+ / B / B+: A mobile dermatoscope device with a contact cap and a rubber attachment, a non-contact cap, a microfiber cleaning cloth, a microfiber device bag and a charging cable.

Device Model U: A mobile dermatoscope device with a contact cap, a universal attachment plate, a non-contact cap, a microfiber cleaning cloth, a microfiber device bag and a charging cable.

Set Up Instructions for Device Models A / A+ / B / B+


These models come in different sizes to fit the specifications of your smartphone/ tablet and are typically recommended for medical professionals or patients looking for an attachment to a specific smartphone/tablet device. Please refer to MetaOptima's website for the most updated list of smartphones and tablets.

To attach the device to your smartphone/tablet, place it over the smartphone camera and firmly squeeze the attachment part together. The device lens should be perfectly aligned with the camera lens. For best results, you should remove any case and screen protectors you have on your smartphone/ tablet.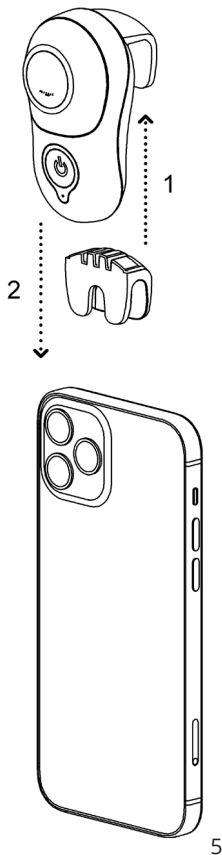

Set Up Instructions for Device Model U

This model is compatible with any smartphone/tablet and recommended for medical professionals or patients with a smartphone/tablet that are not covered by MoleScope models A, A+, B, B+ or want to switch between different smartphone/tablet devices.

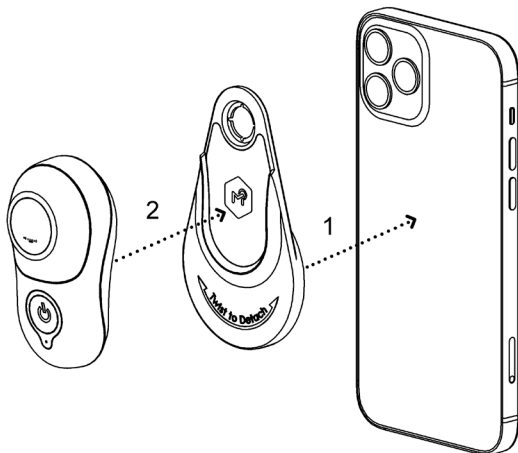
To learn how you can get started, follow the steps below.



Step 1: Twist and lift the lid to reveal the sticky pad underneath. Make sure the sticky pad is clean and free of dust, dirt and oil. If the pad is dirty or losing its adhesive grip, simply wash under water and let air dry prior to your next imaging session.


Do not use hot water, cleaning agents, or a hair dryer to quicken the process. Avoid direct contact with the sticky pad!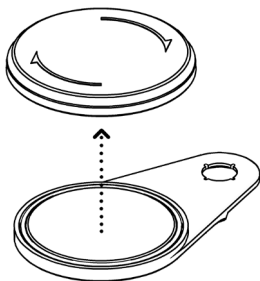

Step 2: Clean the back of your smartphone, tablet, or device case with the provided microfiber cloth to remove any smudges, dirt, or oil. Next, align the guide marks of the Attachment Plate with the center of the smartphone/tablet camera and gently push to secure the plate. If the adhesion is not adequate, you have the option of attaching the plate to your smartphone or tablet case that has a hard and smooth back. Avoid rubber, leather, and textured surfaces. If your smartphone has multiple camera lenses, align the attachment with the center of the main camera. For a reference of popular smartphone model camera placements visit www.molescope.com/molescope2

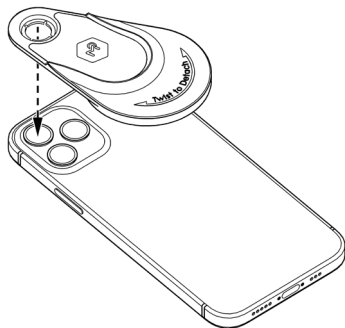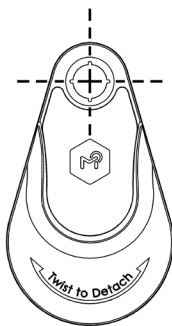
Step 3: Place the MoleScope device on the universal attachment plate. You will hear a clicking noise, indicating that the device is magnetically locked into place. Press the power button to turn on your device and begin your imaging session.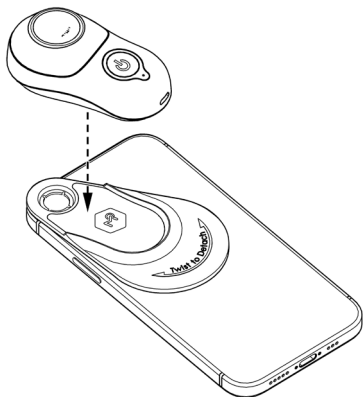

Step 4: When the imaging session is over, pull to remove the MoleScope device, then gently twist the universal attachment plate to detach it from your smartphone or tablet and place the lid back on the sticky pad. Do not grab and lift the plate!

Permanent Attachment to a Smartphone or Tablet Case

If your device will be used frequently, you may peel off the sticky pad from the universal attachment plate and replace with provided double-sided tape. Repeat steps 2 and 3, then simply pull to remove the MoleScope device. Only use the permanent adhesive on the case and do not attach directly to your smartphone or tablet.

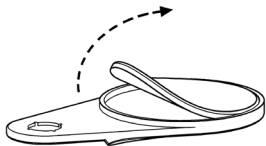Battery and Charging

We recommend that customers fully charge their device before their first use. Complete charging duration is 90 minutes. Do not use the device while it is on charging mode. To conserve energy, this product is designed to automatically turn off after 4 minutes. The battery can be expected to supply multiple sessions. The device has a low battery indicator. If the light indicator is flashing red, the battery is low and it may need to be recharged. To charge, plug the device into a USB port. While charging, the charge indicator will glow red; when fully charged, it will appear green. This product uses a special rechargeable lithium-polymer battery, which is not replaceable.

Cleaning and Disinfection

The exterior and contact plate of your MoleScope should be cleaned by wiping with isopropyl alcohol (70% vol.) or a disinfectant solution in between users. Turn the device off and disconnect it from the charging cable before cleaning and disinfection.

Do not use alcohol or disinfectants in the optical areas of the unit.

Do not use abrasive material on any part of the equipment or immerse the device in liquid. Do not autoclave.

This device lens should be treated as high-quality photographic equipment. It should be cleaned with the provided microfiber cloth or an air duster, and protected from harmful chemicals.

Troubleshooting

Your product is designed for trouble-free operation. There are no adjustments to be made nor is any special servicing required. Never attempt to open the device for any reason.

No Light: Fully recharge the battery and check again. If the condition persists, stop using your device and contact support@metaoptima.com

LED Fails: The LEDs used in your product are designed to last over 50,000 hours. If any one of the LEDs fails, stop using your device and contact support@metaoptima.com

Mechanical Malfunction: Stop using your device and contact support@metaoptima.com

If your product has any other problems or is not working properly, please stop using it and contact us at support@metaoptima.com or contact your local MetaOptima dealer. Please check our website (www.metaoptima.com) for the most current troubleshooting information.

Warranty

Your device has a 2-year warranty. For warranty issues, please contact the reseller or store where you bought the product. The serial number is labeled on the box and back of your device.

Specifications

- 10x Optical magnification with 16 mm wide field of view
- 6 bright white LEDs with polarization rate of 99.97%
- Contact and non-contact imaging caps with easy magnetic attachment
- Anti-reflective glass contact plate with scale
- ON/OFF switch with 4 minutes auto turn off and low-battery/charging indicator
- Rechargeable lithium-polymer battery with 6 hours of working time
- USB 2.0 high speed charging cable
- Smartphone and tablet compatibility

Environmental conditions for operation	0 °C to +45 °C 45 % to 75 % Relative Humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +35 °C 45 % to 75 % Relative Humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-10 °C to +55 °C 45 % to 75 % Relative Humidity 500 hPa to 1060 hPa

Disposal

The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

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Guidance and manufacturer's declaration - electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such environment.

Emission test	Compliance	Electromagnetic environment - Guideline
RF emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, RF-emission is very low and it is unlikely to cause any interference in nearby electronic equipment.
RF emissions CISPR11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Not Applicable	Not Applicable
Voltage Fluctuations / Flicker Emissions IEC 61000-3-3	Not Applicable	Not Applicable

Guidance and manufacturer's declaration - electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should assure that it is used in such environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - Guideline
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	Not Applicable	Not Applicable	Not Applicable
Surge IEC 61000-4-5	Not Applicable	Not Applicable	Not Applicable
Voltage dips, short interruptions, and voltage variations on power supply input lines IEC 61000-4-11	Not Applicable	Not Applicable	Not Applicable
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	Not Applicable	Not Applicable	Not Applicable

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - Guideline
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV contact discharge ± 8 kV air discharge	Recommended separation distance: $d = 3,5/3 * \text{SQRT}(P/W)$ $d = 3,5/3 * \text{SQRT}(P/W)$ 80 MHz to 800 MHz $d = 7/3 * \text{SQRT}(P/W)$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

A. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

B. Over the frequency range 150kHz to 80 MHz, field strengths should be less than 3 V/m.

Note 1: At 80Hz and 800MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Symbols Glossary

	Caution		CE marking
	Consult instructions for use		Fragile, handle with care
	Non-sterile		Temperature Limit
	Manufacturer		Humidity limitation
	Date of manufacture		Atmospheric pressure limitation
	Do not dispose as municipal waste		Keep dry
	Do not use if damaged		Recycling
	Follow instructions for use		Serial number
	Non-ionizing electromagnetic radiation		Unique device identification
	UK conformity assessed marking		EN: Medical Device FR: Dispositif médical ES: Producto sanitaria NL: Medisch hulpmiddel DE: Medizinprodukt IT: Dispositivo medico
	Authorized representative in the European Community		

MetaOptima Technology Inc.
Suite 1050, 1185 W Georgia Street,
Vancouver, BC, Canada V6E 4E6
www.metaoptima.com
Tel: +1-778-328-194

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Australia Sponsor

MetaOptima Australia Pty. Ltd.
Level 3, 223 Liverpool Street,
Darlinghurst, NSW, 2010, Australia

UK Responsible Person

Emergo Consulting (UK) Limited,
c/o Cr360 – UL International,
Compass House, Vision Park Histon,
Cambridge CB24 9BZ, United Kingdom